



Get Started

Accessing Your Diagnocat Account

- 1. Open Your Browser: Launch Google Chrome for optimal performance.
- 2. Visit Our Website: Navigate to www.diagnocat.com.
- 3. Select Your Region: Choose your region to ensure localized support.

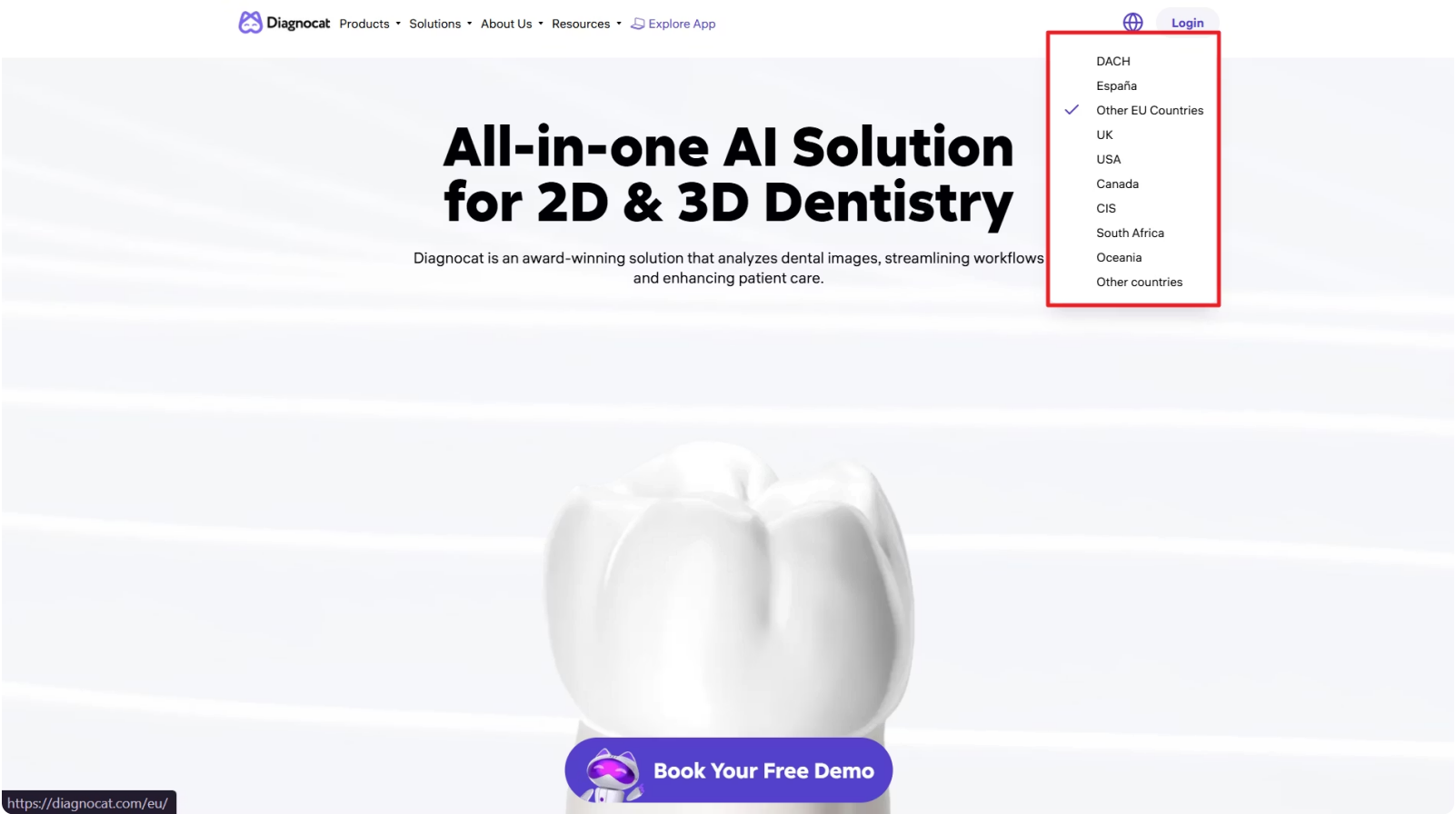


- 4. Log In: Click on 'Login' and enter credentials: Input your email and password. If you need to reset your password, use the link provided on the login page.

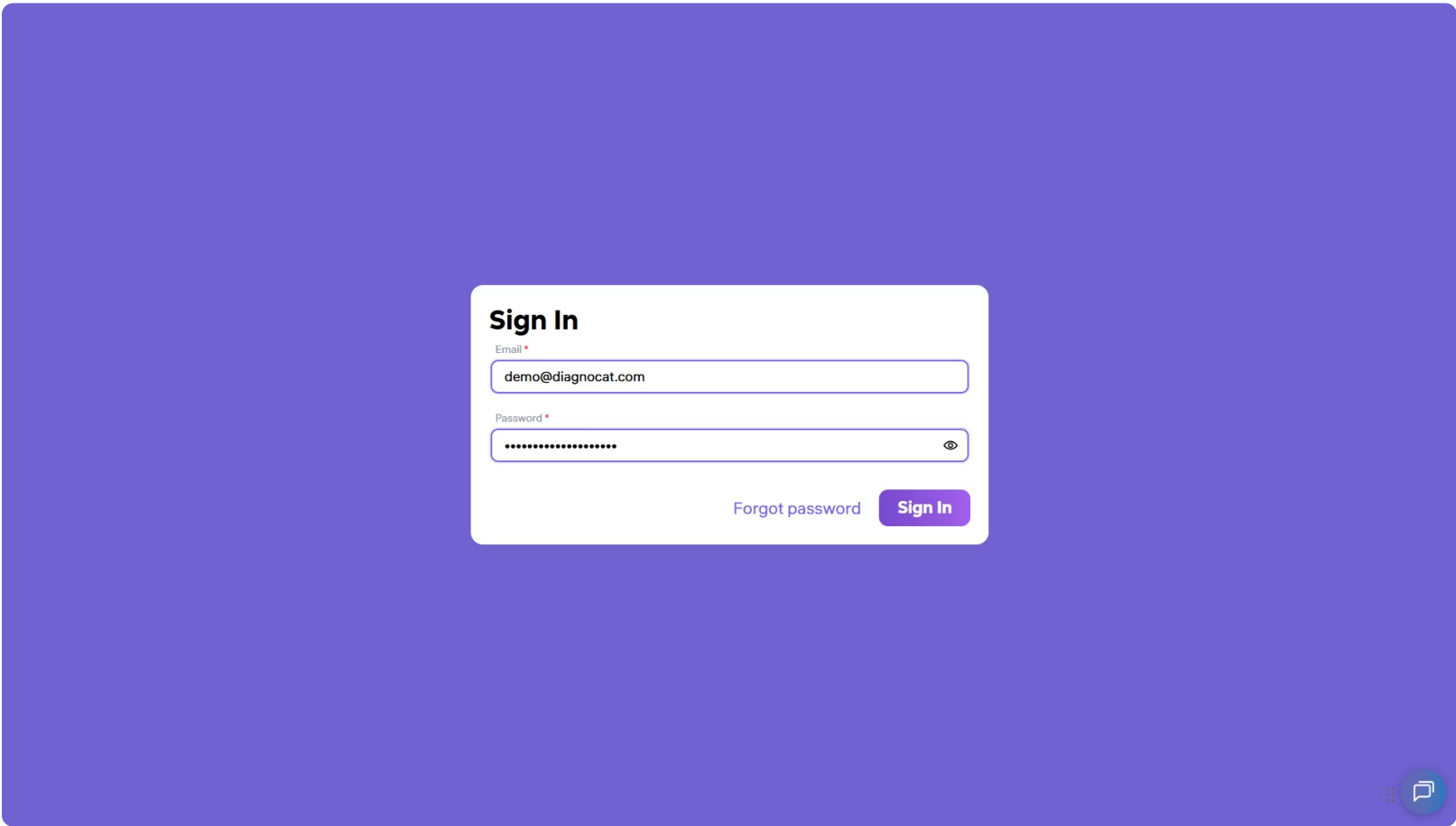


Navigating the Homepage


Patient List: Centralized on your screen, this list allows quick access to patient records.

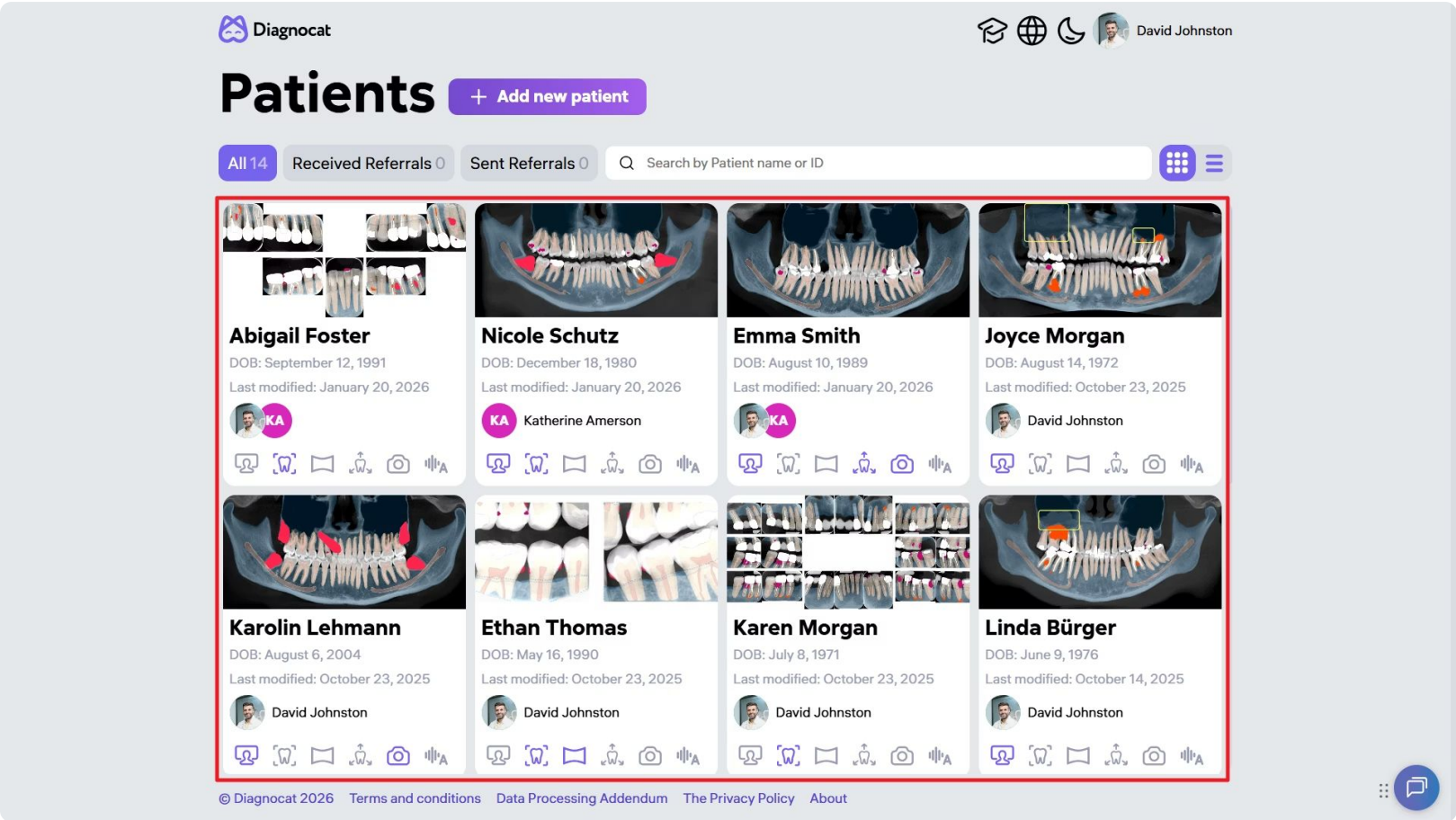

Search Functionality: Use the search bar at the top to find patients by name, or ID.

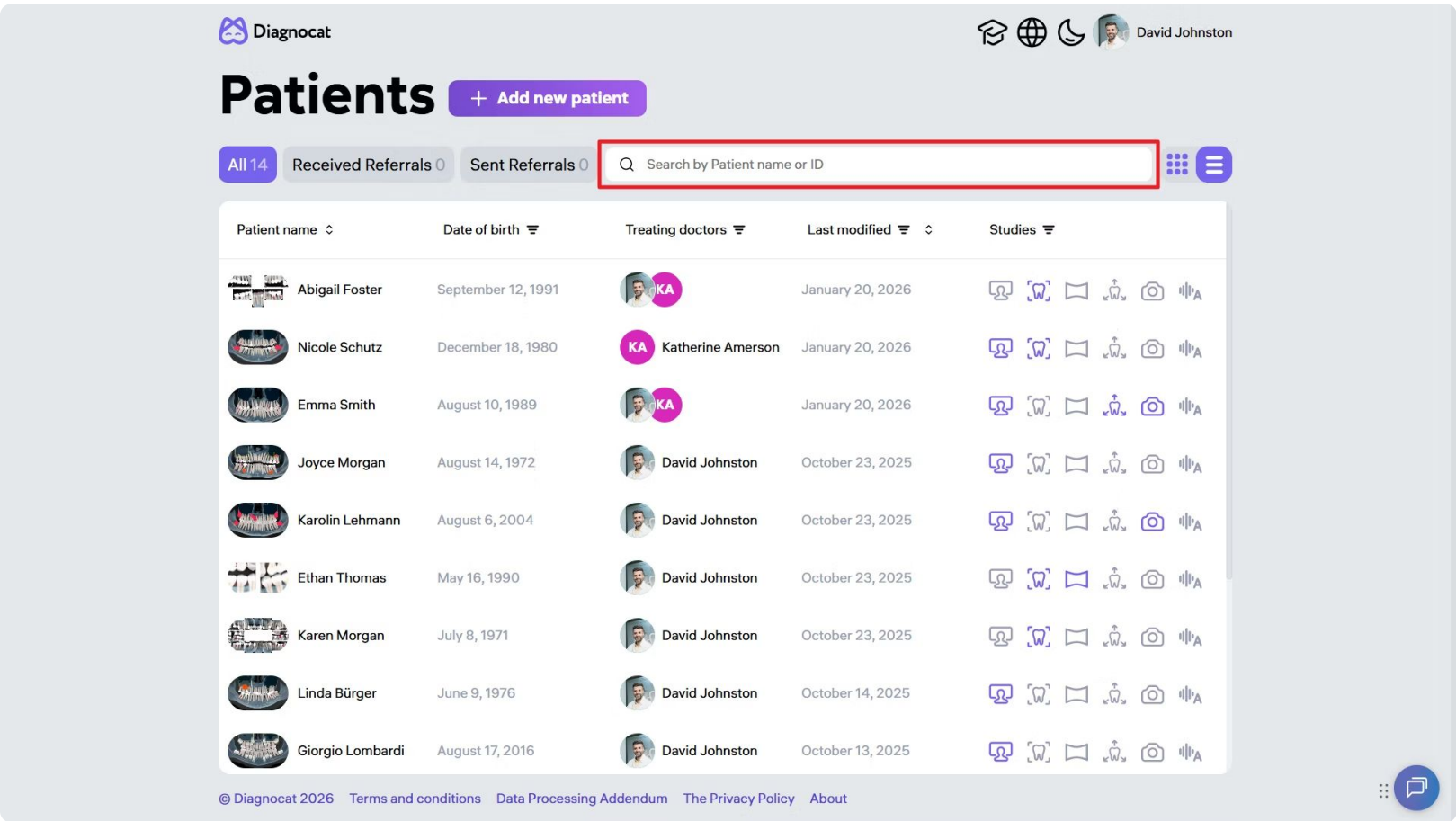

Layout Options: Switch between list and matrix views for your convenience.

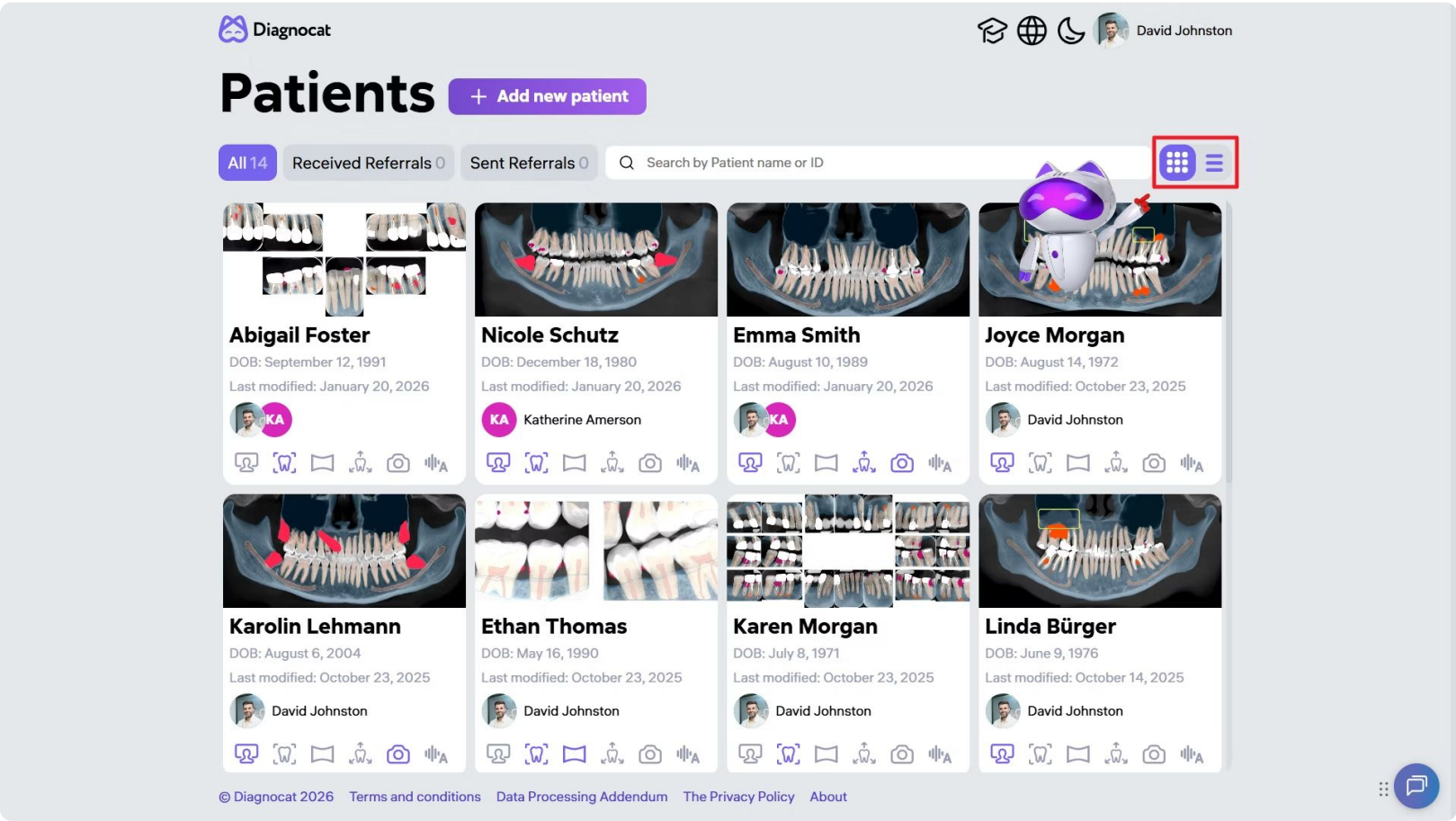

In the upper right corner, you will see a set of buttons.




Graduation Cap Icon

Opens the Knowledge Base with guides, videos, and updates.




Globe Icon

Selects the interface language.

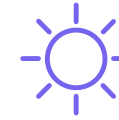

Sun/Moon Icon. Theme Toggle

Switches between dark and light modes.

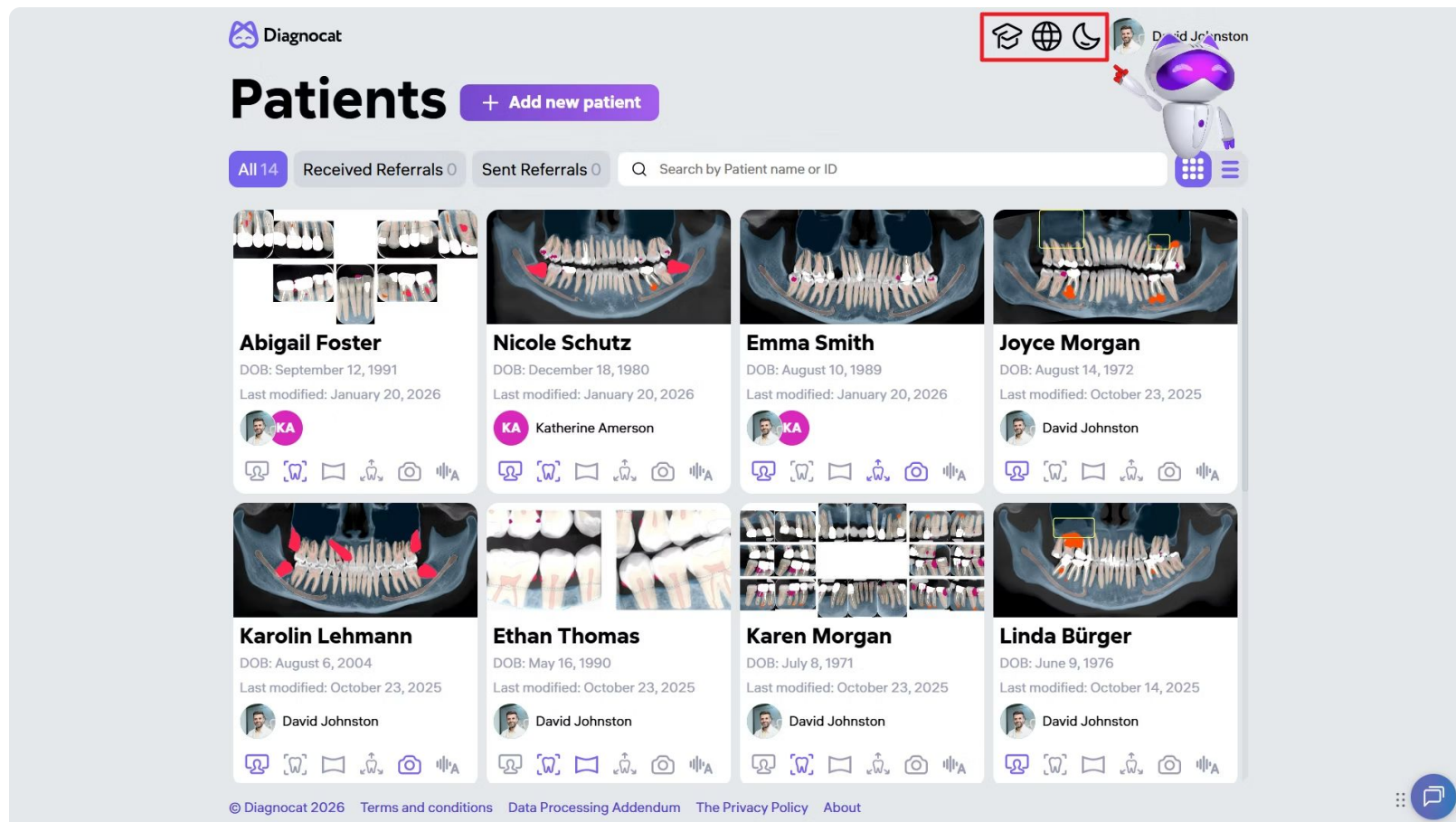

The screenshot displays the Diagnocat Patients interface. At the top right, a red box highlights three icons: a graduation cap, a globe, and a sun/moon symbol, corresponding to the features described in the text. The interface includes a search bar, a grid of patient cards with dental X-rays and photos, and a footer with copyright and policy links.

Diagnocat

Patients

+ Add new patient

All 14 Received Referrals 0 Sent Referrals 0

Search by Patient name or ID

Abigail Foster
DOB: September 12, 1991
Last modified: January 20, 2026
Katherine Amerson

Nicole Schutz
DOB: December 18, 1980
Last modified: January 20, 2026
Katherine Amerson

Emma Smith
DOB: August 10, 1989
Last modified: January 20, 2026
Katherine Amerson

Joyce Morgan
DOB: August 14, 1972
Last modified: October 23, 2025
David Johnston

Karolin Lehmann
DOB: August 6, 2004
Last modified: October 23, 2025
David Johnston

Ethan Thomas
DOB: May 16, 1990
Last modified: October 23, 2025
David Johnston

Karen Morgan
DOB: July 8, 1971
Last modified: October 23, 2025
David Johnston


Linda Bürger
DOB: June 9, 1976
Last modified: October 14, 2025
David Johnston

© Diagnocat 2026 Terms and conditions Data Processing Addendum The Privacy Policy About


Creating a New Patient Profile

If Diagnocat is integrated with your Practice Management Software (PMS), new patient profiles are automatically created. For manual creation or if integration isn't available:

1. Add Patient: Click on the 'Add Patient' button.

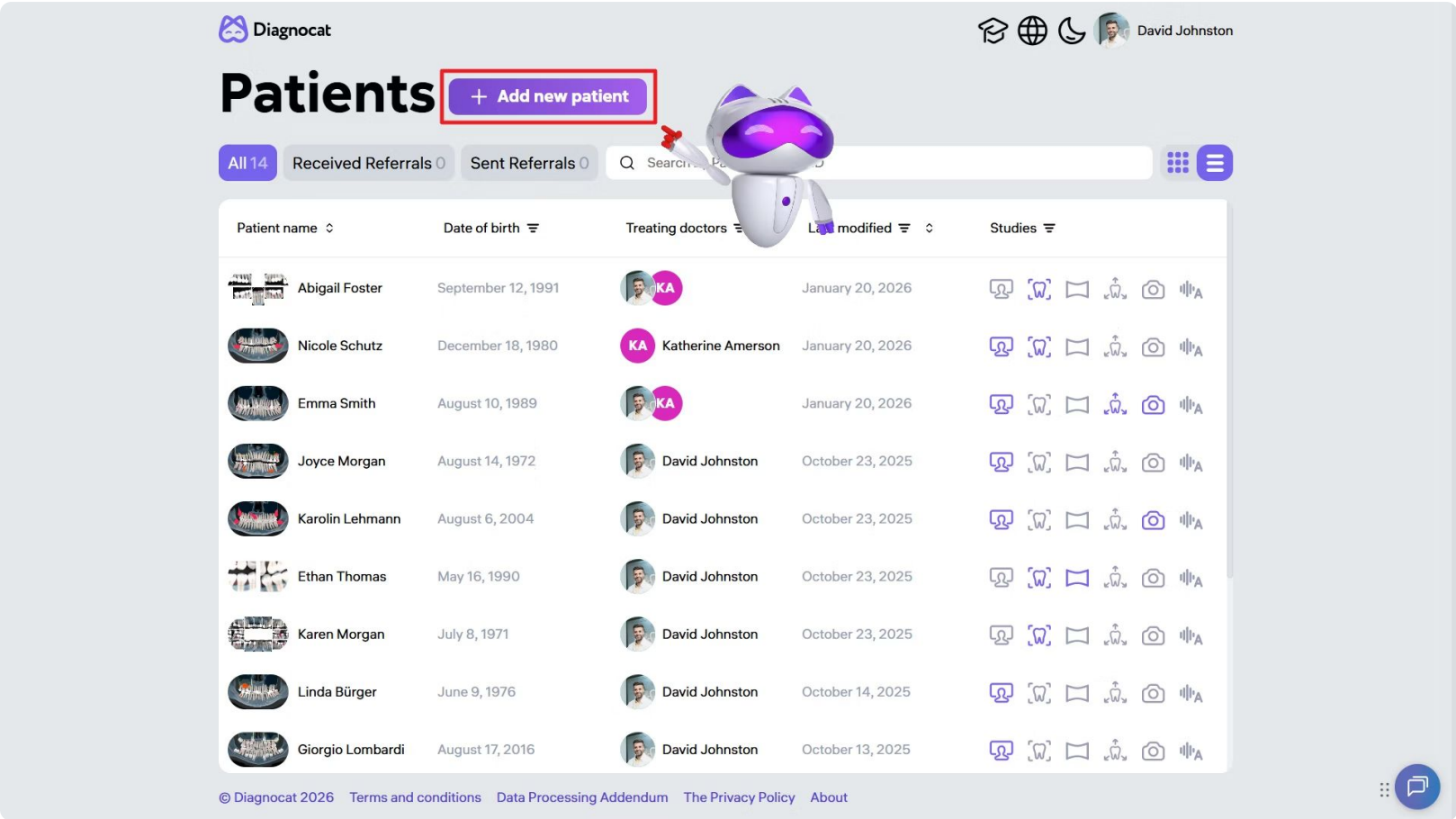

2. Enter Details: Provide the patient's first and last name, email (optional), date of birth, and assign a treating doctor.
3. Assign ID: Input a unique patient number for easy tracking.
4. Save Profile: Click 'Add' to add the new patient to your list.

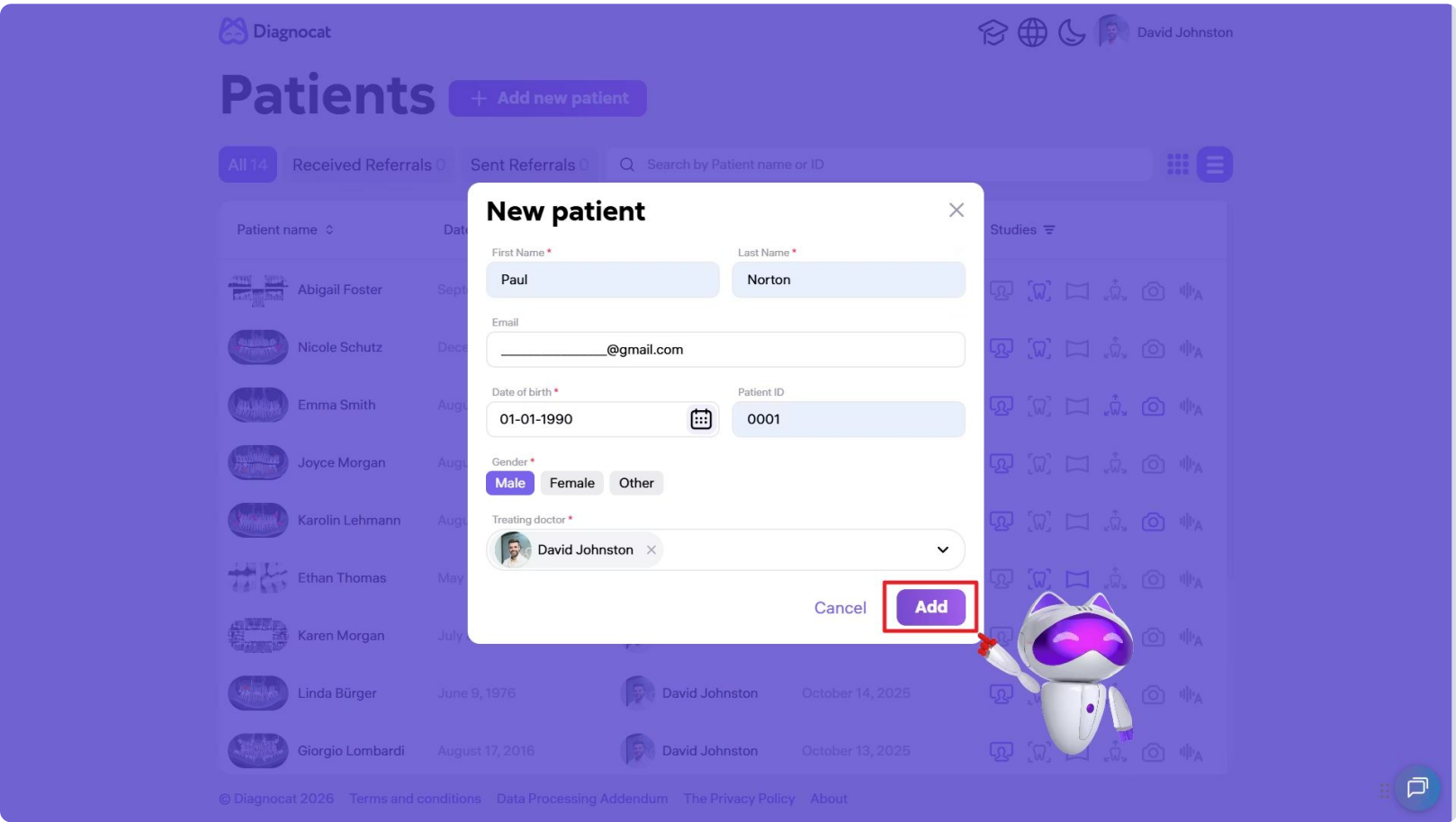

Organizing Patient Records

Sort the patient list by criteria such as name, date of birth, study type, last modified or attending doctor to streamline navigation.

 Diagnocat

 David Johnston

Patients

+ Add new patient

All 14Received Referrals 0Sent Referrals 0

Search by Patient name or ID

Patient name

Date of birth

Treating doctors

Last modified

Studies

☐ Adults

☐ Children

Clear

Apply

Select...

☐ David Johnston

☐ Katherine Amerson

Clear

Apply

☒ All

☐ Today

☐ One Week

☐ Custom

Clear

Apply

☐ CBCT

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
☐ IOXRy

☐ Pano

☐ Intraoral Scan

Clear

Apply

 Ethan Th...		 David Johnston	September 29, 2025	
 Veronica		 Katherine Amerson	January 20, 2026	
 Emma Sn...		 David Johnston	October 13, 2025	
 Lukas Shuster	February 9, 2016	 David Johnston	October 23, 2025	
 Nicole Schutz	December 18, 1980	 David Johnston	October 23, 2025	
 Alessandro Rossi	September 13, 1978	 David Johnston	October 13, 2025	
 Karen Morgan	July 8, 1971	 David Johnston		
 Joyce Morgan	August 14, 1972			
 Giorgio Lombardi	August 17, 2016			

© Diagnocat 2026

Terms and conditions

Data Processing Addendum

The Privacy Policy

About